

# Lifenaxx® \* User Manual

## Wlan Otoscope LX-032

The Declaration of Conformity for this device is under the Internet link: [www.technaxx.de/](http://www.technaxx.de/) (in bottom bar "Konformitätserklärung"). Before using the device the first time, please read the user manual carefully. Service phone No. for technical support: 01805 012643 (14 cent/minute from German fixed-line and 42 cent/minute from mobile networks).

Free Email: [support@technaxx.de](mailto:support@technaxx.de)

Keep this user manual for future reference or product sharing carefully. Do the same with the original accessories for this product. In case of warranty, please contact the dealer or the store where you bought this product.

**Warranty 2 years**

This product is intended to use on humans and animals. The product may only be used for cleaning and checking the auditory canal or nasal cavity. When using the device, make sure that you do not penetrate too deeply into the openings, you could cause damage.

### Features

- CMOS Camera 3MP.
- Resolution 1920 x 1080P 30FPS.
- 6 Axis Gyroskop
- Focus distance 1,5-2cm
- Battery Low indicator



### Technical specifications

USB (load in dry condition only)	5V max. 0,2A
Battery	3,7V / 250mAh
Camera	3MP 1920x1080 30FPS
Working time	60 min.
Loading time	90 min.
WLAN	Transmitting distance ~8m (open space) Frequency band 2.4GHz Radiated transmission power max. 20mW
LED	6
Diameter of the Lens	3,9mm
Dimension	Ø1.3x 14cm
Weight	17g









### First use

- Load the device fully with the USB cable, this may take up to 90 minutes.
- Open

the APP store and search for iTiMO APP. ●Install the APP on your device and open the APP.

●Start the Otoscope. ● Press the Symbol . ●A new Window open all available Wlan connections (Android). For IOS devices pls go to Settings, Wi-Fi. ● Chose the iTiMO-xxxx device, you may get a warning that no internet service is available on this network. ●Then go back to the iTiMO APP and press the  button again you will see the live video from the Otoscope camera now. ●Attache the cleaning spoon of your choise and start cleaning carefully. ●We suggest to fit the Phone/ Tablet in a Holder in direct view.

### APP

	Freeze the actual picture
	Start the live video
	Take picture
	Start or stop the video recording
	Open the galary, here you will find the recorded photos and videos.
	bring you one step back
	Settings
	Change from right to left side

### Normal use

- The otoscope should only be inserted into the auditory canal when a spoon is fitted.
- Place the spoon on the otoscope so that the spoon fits tightly.
- be carefull auditory canal and nose are sensitive.

### Cleaning

Before and after use you should clean the camera.


### Safety Instructions

→ Do not look into the LED light. → Only recharge with supplied or compatible USB cable. → This device is not intended for use by children or persons with limited mental capacity or lacking experience and/or lacking expertise. → This device is not intended for commercial use.→ Any other use or modification of the device is considered improper and

involves significant risks. →The manufacturer assumes no liability for damages due to improper use. Check the correct operation of the device before use! →Do not use the device if there are visible signs of damage. →Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments). →This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners. →Do not modify the device. → Never leave the device charging unattended.

### **WARNINGS & CAUTION:**

For adult use only, do not use the device in the presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide, or near flammable substances, vapors or gases. Do not operate the device near heating openings or in direct sunlight. Keep the device dry. Do not immerse the device in water.

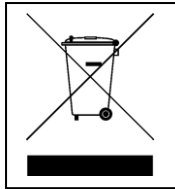

**Hints for Environment Protection:** Packages materials are raw materials and can be recycled. Do not disposal old devices or batteries into the domestic waste. **Cleaning:** Protect the device from contamination and pollution (use a clean drapery). Avoid using rough, coarse-grained materials or solvents/aggressive cleaner. Wipe the cleaned device accurately. **Distributor:** Technaxx Deutschland GmbH & Co.KG, Kruppstr. 105, 60388 Frankfurt a.M., Germany