NEWSTAR

AV & IT MOUNTS

MED-M200

UK Medical screen mount

DE Medische monitor steun

DE Medical Display Halterung

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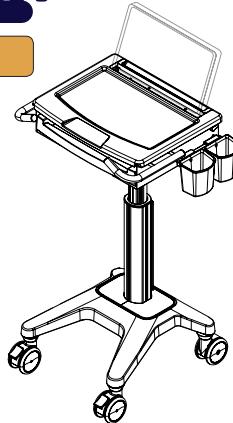
ES Soporte para pantalla medica

NO Medicinske skærbeslag

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
Features

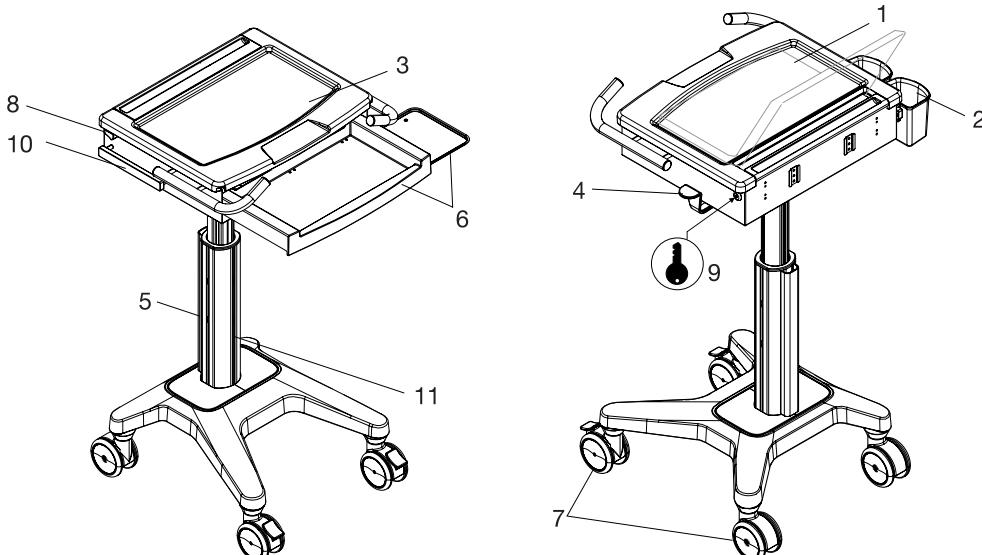
1. Laptop compartment
2. Storage basket
3. Work surface
4. Lift brake release (release brake to raise or lower the cart)
5. Cable management
6. Sliding keyboard tray with mouse holder and sliding mouse tray (to accommodate both right and left hand users)
7. Casters (2 rear non-locking and 2 front locking)
8. Latch
9. Key and lock hole
10. Hanger for accessories
11. Track for accessories mounted



Index

Features	p2
Adjustment direction	p3
Checklist	p3
Assembly instruction	p4
Cart's adjustment and operation	p7
Summary of warnings	p8
Maintenance	p10
Troubleshooting	p11

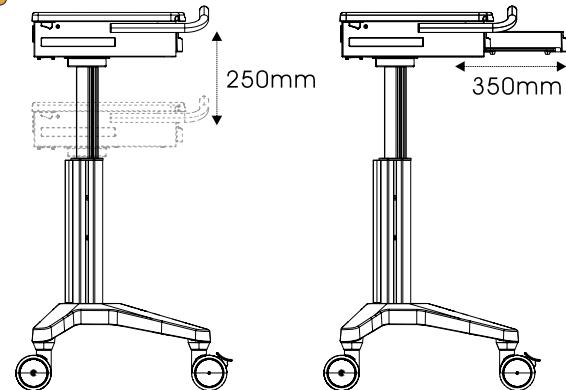Adjustment direction



Checklist

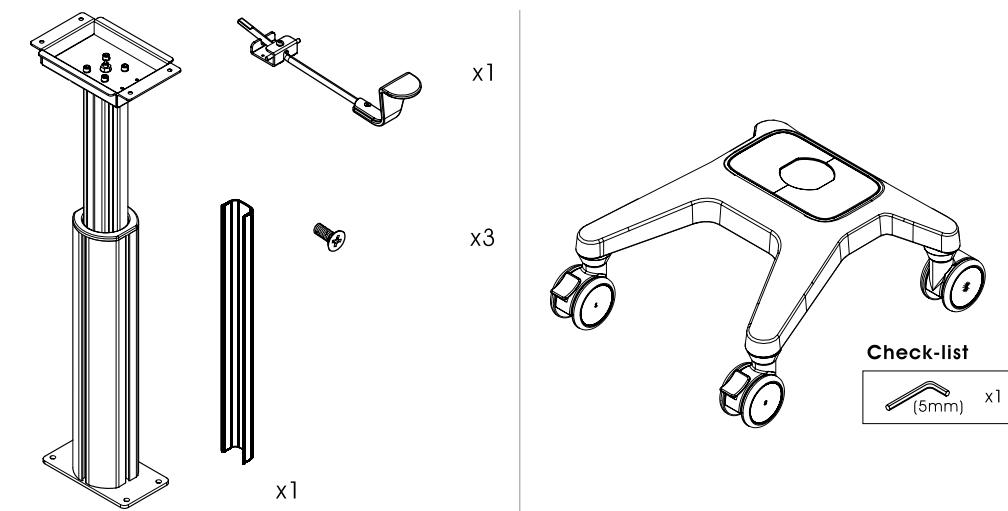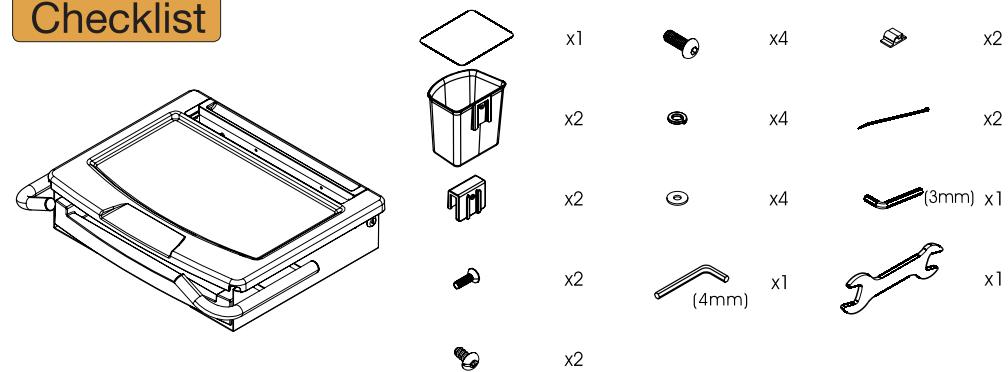
Assembly instruction

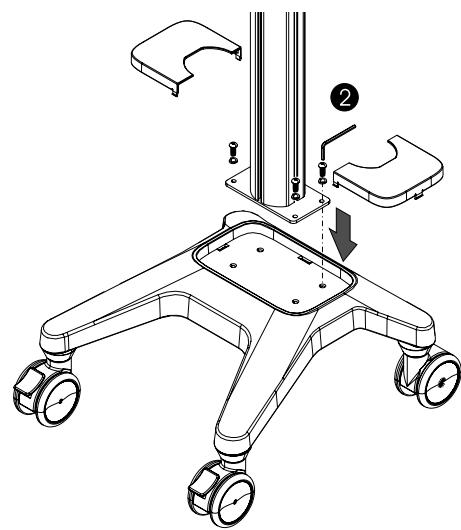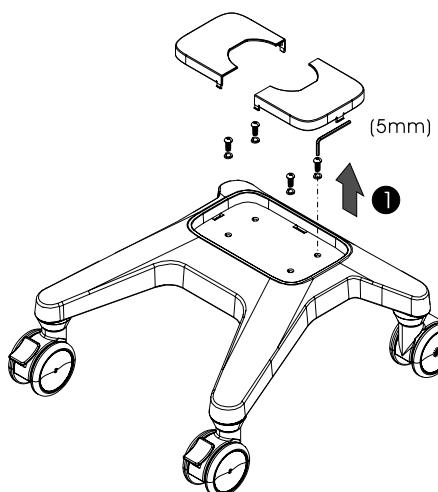Step 1


Step 3


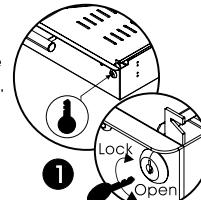
(1) Unlock the work surface by turning the key counterclockwise.

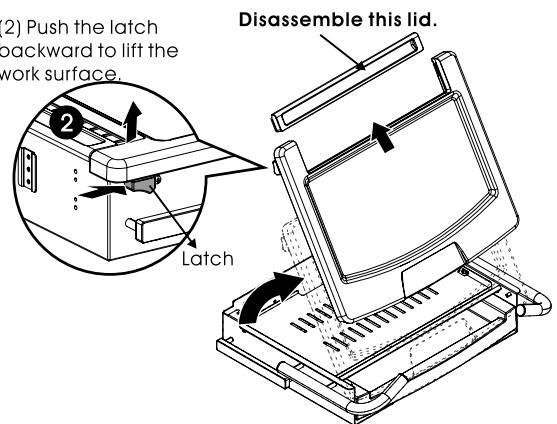(2) Push the latch backward to lift the work surface.

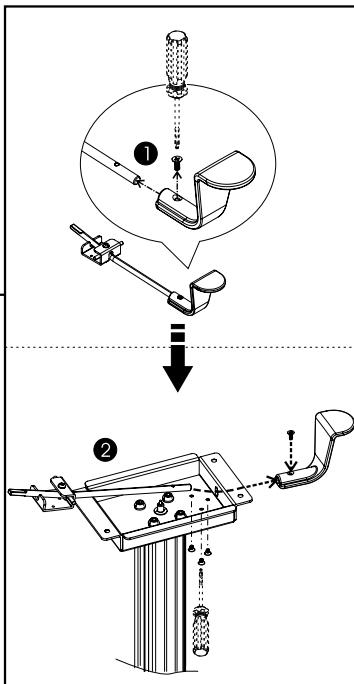
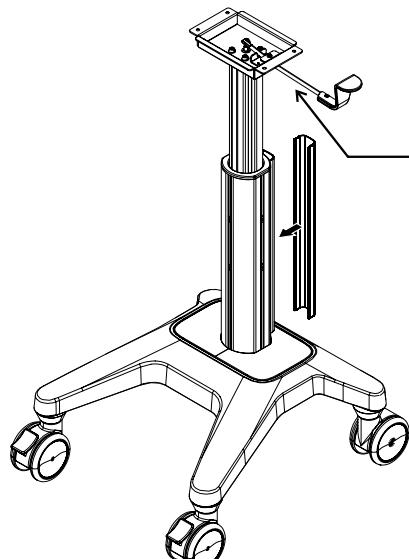
Step 2


Step 4

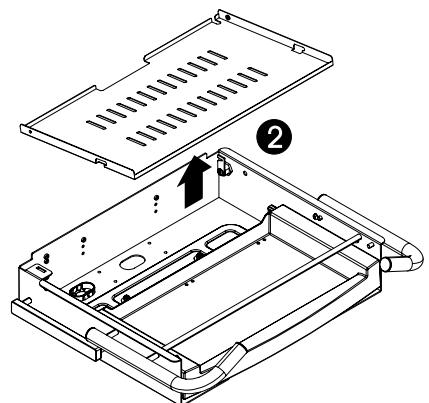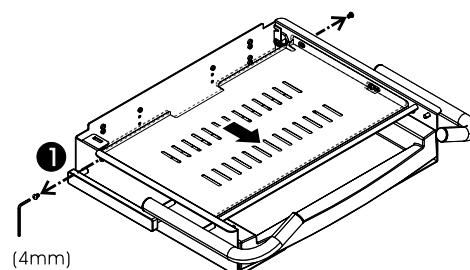Use a 4mm allen key to loose the screw. Push the cooling pad forward to take it out.


Step 5

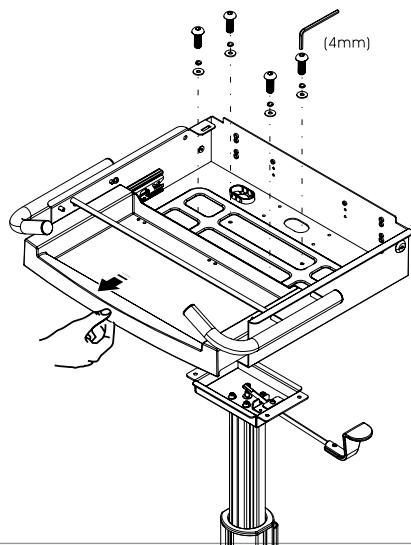Use the screws to combine the top plate and the middle pillar.


Step 6

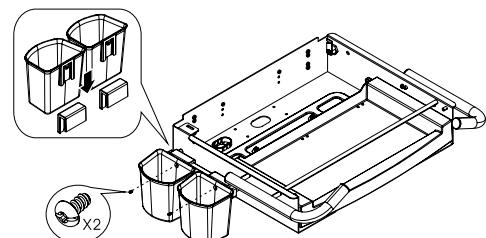Storage basket can be placed as different demands.


OR

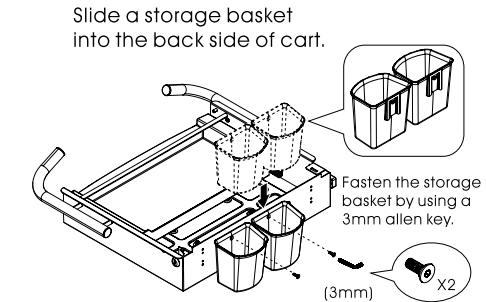
Step 8

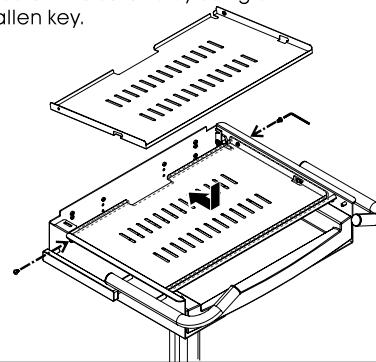Put back the cooling pad and fasten the screws by using a 4mm allen key.

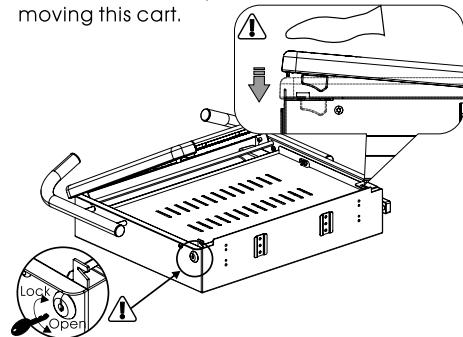
Set the laptop into the compartment and then replace the work surface. Please notice the work surface must be locked and latch must be pushed forward before moving this cart.

Step 7

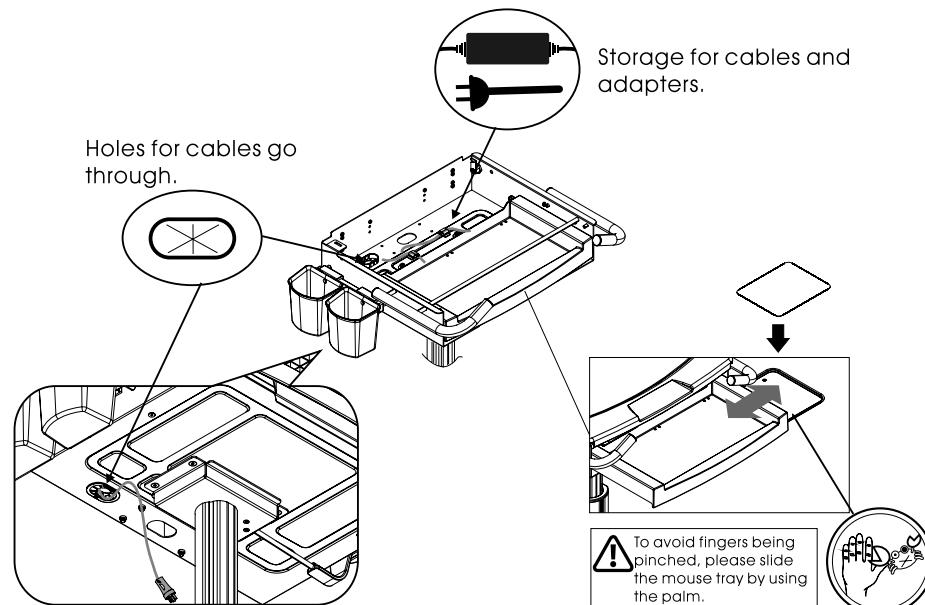Direction of cable management and accessory assembly




Cart's adjustment and operation

In case of users harm or cart damage, do not adjust lift mechanism with an empty cart which may cause work surface to rise speedily.

Notice:

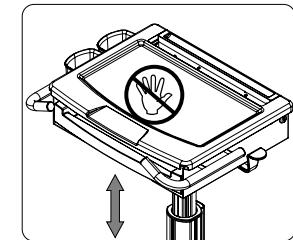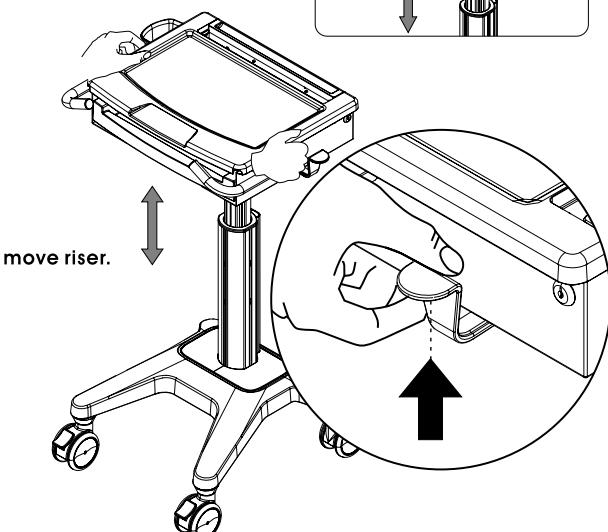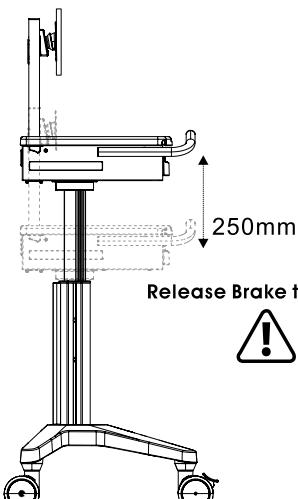
While releasing brake to move the height of work surface riser or lower, please put two hands on two sides of work surface as figure. Do not push the center of work surface for lower adjustment. Please use two hands to uphold slightly for riser adjustment if the work surface is still motionless.

Summary of warnings


⚠ WARNING

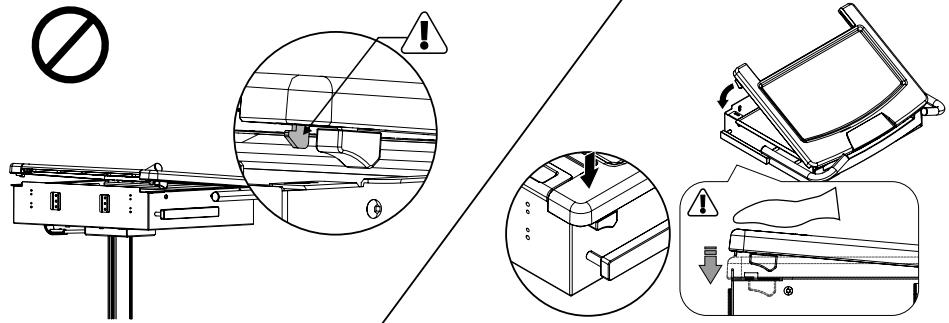Please notice that latch must be pushed forward after replacing the work surface.


⚠ WARNING

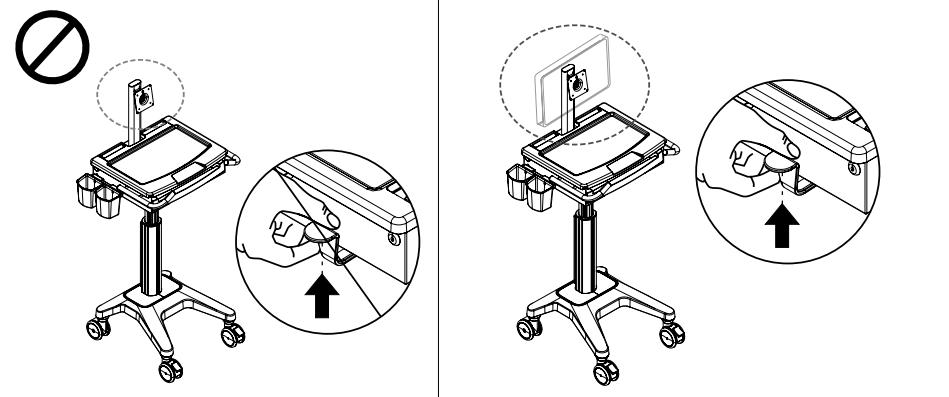In case worksurface goes up speedily, please install the monitor properly before adjusting lift mechanism.

⚠ WARNING


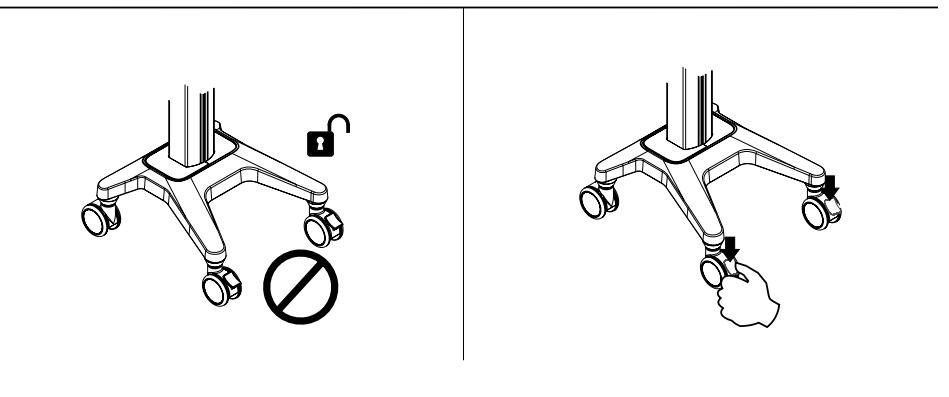
When the cart is in motionless, please brake the two front casters.

⚠ WARNING

Notice:

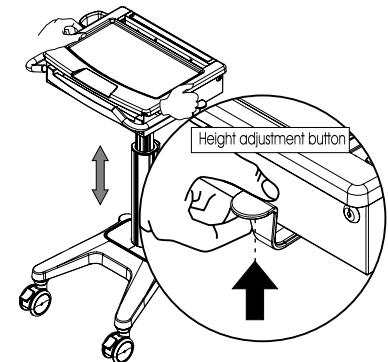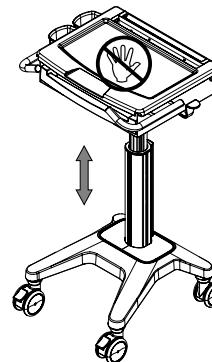While releasing brake to move the height of work surface riser or lower, please put two hands on two sides of work surface as figure. Do not push the center of work surface for lower adjustment. Please use two hands to uphold slightly for riser adjustment if the work surface is still motionless.


⚠ WARNING

Please do not press the whole work surface especially the center area except height adjustment.

⚠ WARNING

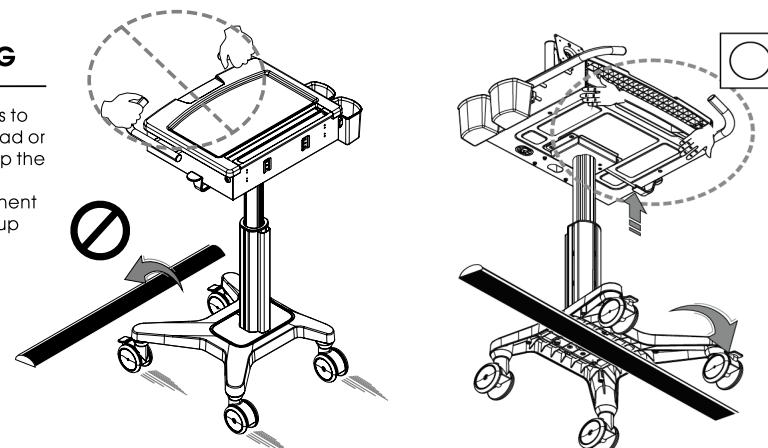When the cart needs to go across bumpy road or barriers, please lift up the cart from bottom of keyboard compartment as figure. Do not lift up from handles.

Maintenance

All cart maintenance must be carried out at least once a month by a professional technician.

Cleaning Warning

▲ Because the cart is under the environment of electrical equipment, inflammable cleansers are forbidden using strictly.

- (1) Before cleaning the cart, please confirm the cart is unplugged from the mains and switched off.
- (2) Please confirm the cart is entirely in dry status before plugging the power into socket.
- (3) While cleaning the cart, please use damp cloth to wipe off the superficial cleansers until completely dry.
- (4) The cart is not only forbidden in moist environment but also forbidden liquids running into the cart.
- (5) To avoid surface finish being damaged, do not use any abrasive materials for clean.
- (6) Before using any cleanser on the cart, please test it on a small part to see whether the surface is damaged or not.

Please be cautioned that these guidelines can not ensure infection control.

Please kindly consult hospital's infection control consultant for cleaning advice.

Cleaning Recommendations

- (1) Please use non-abrasive or diluted products to clean plastic components such as soap、water and diluted alcohol.
 - (2) If there are spots of pen or marker on the cart's surface, a soft cloth with 91% isopropyl alcohol is suggested for clean.
 - (3) If there are iodine spots on the cart's surface, a soft cloth with above cleaners is suggested for clean.
- ▲ The following chemicals are forbidden strictly cleaning the cart : Abrasive cleansers、Acetone、Mineral spirits、Paint thinner or any other harsh or toxic chemicals.

Inspections

All cart inspections must be carried out at least once a month by a professional technician.

In case of faults or damage occurring, stop operating this cart and contact distributors for further assistance instantly.

Caster

To allow cart move as intended, please inspect the casters regularly and keep the cart away from destroyed and untidy ground.

The maximum weight of the cart is 40 kg.

(Loading limitation of accessories, please see title page.)

Fasteners

If there are any fasteners turning into loose and need to be tightened, do not screw fasteners over tightly. Over tighten fasteners/screws might cause not only screws but also components damage.

Troubleshooting

Cart is difficult to move

Ensure the two front locking casters are not locked.

Inspect whether there are any debris sticking to casters.

Cart is difficult to adjust height

1. When the lift mechanism is unable for lift and descent, ensure the lift brake is pressed completely.
2. And then lift the work surface up slightly to prevent work surface rising rapidly.
3. If lift mechanism still doesn't work, please contact distributors for further assistance.
4. If the lift mechanism keeps going up automatically, please contact distributors for further assistance.

EN Caution

- To ensure safety, please read this manual carefully before installation and follow the instructions. Store this manual in a secure place for future reference.
- The manufacturer shall not be legally responsible for any equipment damage or personal injury caused by incorrect installation or operation other than that covered in this manual.
- The mount is designed for easy installation and removal. The manufacturer shall not be liable for damage to equipment or personnel injuries arising out of human factors or acts of God, such as earthquake or typhoon.
- It is recommended that the Wall mount bracket be installed by qualified personnel only.
- At least two persons are needed to install or remove the product to avoid hazard of falling objects.
- Please carefully inspect the area where the wall mount is to be installed:
 - Avoid placing it near to water sources, such as pipes, humidity or contact with water,
 - Do not install the product near air conditioning vents or areas with excess dust and fumes,
 - Only install on vertical walls and avoid slanted surfaces,
 - Do not install in places subject to any shock or vibration,
 - Do not install in places subject to direct exposure to bright light, at it may cause eye fatigue when viewing the display panel.
- Maintain sufficient space around the display to ensure adequate ventilation.
- To ensure safe installation, first check the structure of the wall and select a secure mounting location.
- The wall shall withstand enough force to hold at least four times the weight of the Wall mount bracket combined. The mounting location must be able to withstand earthquake or other strong shock.
- Do not modify any accessories or use broken parts. Contact your dealer with any questions.
- Tighten all screws (do not exert excessive force to avoid breaking the screw or damaging its thread).
- Drill holes and bolts will be left in the wall once the Wall mount bracket are removed. Stains may occur after extended use.
- Since the manufacturer has no way to control the wall type and installation of wall mount, the warranty of the product shall only cover the body of the wall mount. The warranty period of the product is 5 years.
- Please consult the English language manual for any dispute on conditions.

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