

Ergotron Carts

Cleaning recommendations



Ergotron carts deliver more than compact, mobile data entry at the point of care. Unparalleled features and superior construction help increase accuracy and efficiency—now and when technologies change. StyleView® and Neo-Flex® workstations are an economical investment built to exceed industry expectations.

⚠ Recommended cleaning steps

The following procedures are not guaranteed to control infection. The hospital infection control administrator or epidemiologist should be consulted regarding cleaning procedures and processes.

- ⚠ To avoid risk of electric shock, do not expose electrical components to water, cleaning solutions or other potentially corrosive liquids or substances.
- ⚠ Do not immerse cart or cart components in liquid or allow liquids to flow into the cart. Wipe all cleaners off surface immediately using a damp cloth. Thoroughly dry surface after cleaning.
- ⚠ Do not use flammable cleaners on cart surfaces due to close proximity of electrical power and equipment.

All paints and plastic cart components will withstand cleaning by most commonly used, diluted, non-abrasive solutions such as quaternary ammonia compounds, ammonia enzyme cleaners, bleach or alcohol solutions.

- Pen and permanent and dry erase markers can be removed with 91% isopropyl alcohol and a soft cloth.
- Iodine stains can be removed with commonly used cleaners and a soft cloth.
- Never use steel wool or other abrasive materials that will damage the surface finish.
- Do not use strong solvents such as trichloroethylene and acetone. These solvents will damage the surface finish.

It is recommended that any cleaning solution be tested on a small, inconspicuous area to ensure surface is not harmed.

© 2009 Ergotron, Inc. rev. 09/22/2009 Content is subject to change without notification

Americas Sales and Corporate Headquarters

St. Paul, MN USA
(800) 888-8458
+1-651-681-7600
www.ergotron.com
sales@ergotron.com

EMEA Sales

Amersfoort, The Netherlands
+31 33 45 45 600
www.ergotron.com
info.eu@ergotron.com

APAC Sales

Tokyo, Japan
www.ergotron.com
apaccustomerservice@ergotron.com

Worldwide OEM Sales

www.ergotron.com
info.oem@ergotron.com

