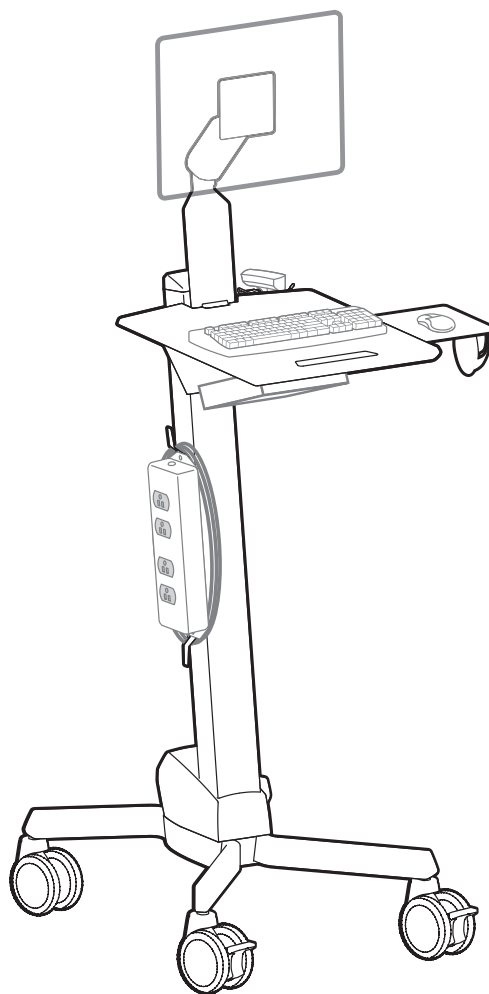
# ergotron®

---

## Neo-Flex™ LCD Cart

---

**EN** User Guide

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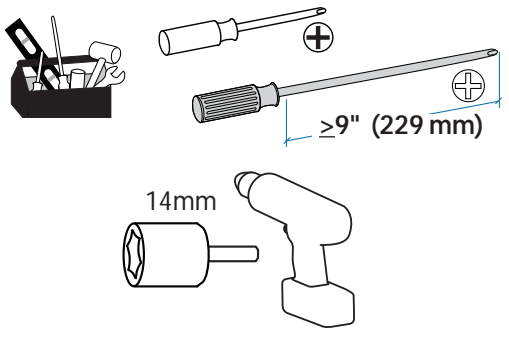
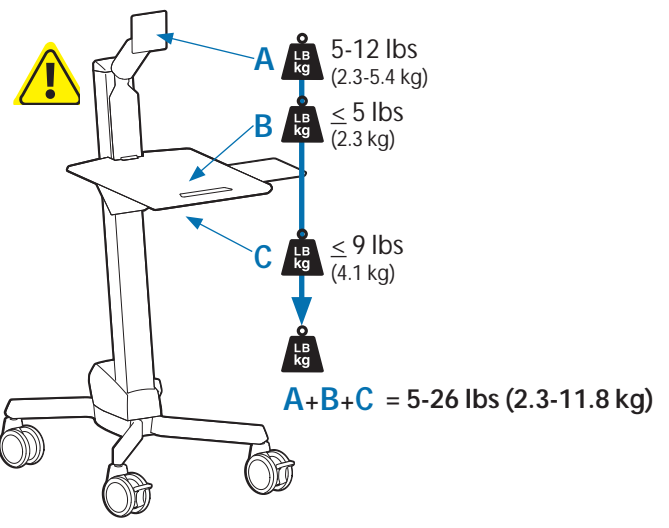
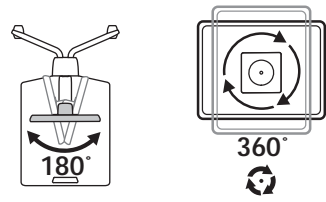
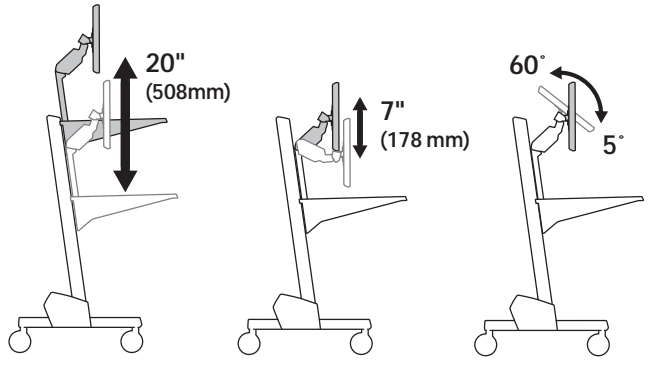
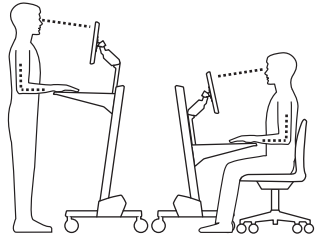
[www.ergotron.com](http://www.ergotron.com) | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: [japansupport@ergotron.com](mailto:japansupport@ergotron.com)



---


	A	B	C
1	1x	1x  1x	4x  M5 x 16mm
2		1x	4x  M5 x 20mm
3		1x	2x  M5 x 20mm
4		2x  M4 x 10mm	Security Screw Tornillo de seguridad Vis de sécurité Sicherheitsschraube mit Innenfünfkant Veiligheidsleutel
5		4x  M4 x 10mm	Güvenlik Vidası 固定ネジ 安全螺丝 Security Screw(안전 스크루)
		8x  M4 x 10mm	1x
			5-sided Security Wrench Llave de seguridad pentagonal Clef Allen de sécurité Fünfkant-Inbusschlüssel 5-kantige veiligheidsleutel 5 Kenarlı Güvenlik Anahtarı 5角固定用レンチ 5角安全扳手 5면 Security Wrench(안전 렌치)
6	1x	1x	
7			
8	4x		1x
9	1x		1x
10	2x	2x	1x
11	4x  2x	8x	4x







1





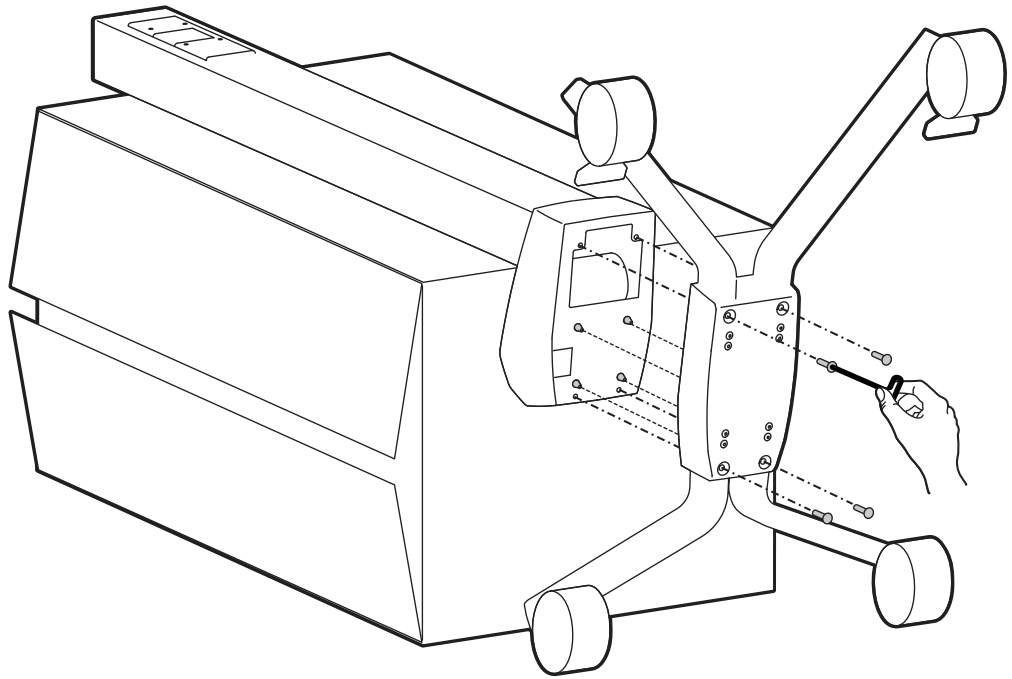
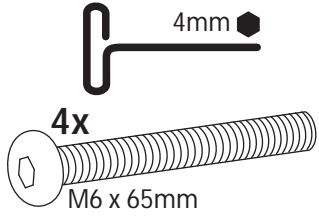
2



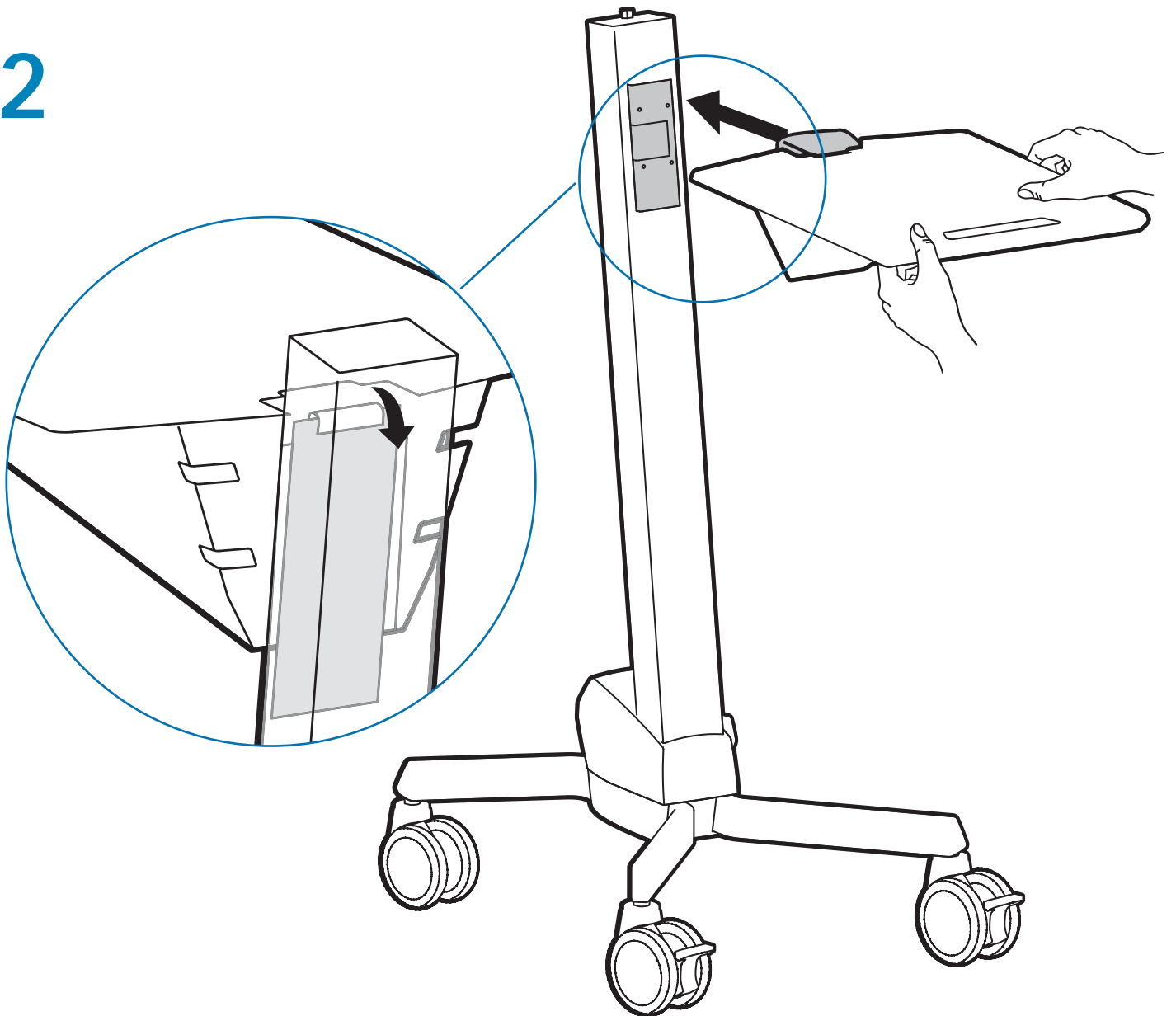
# 3

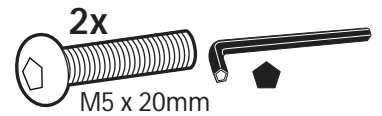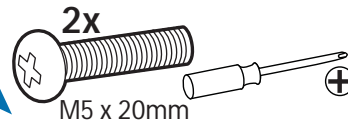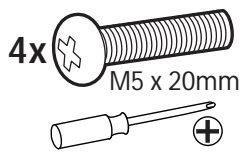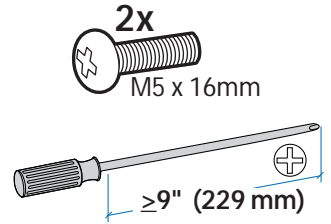
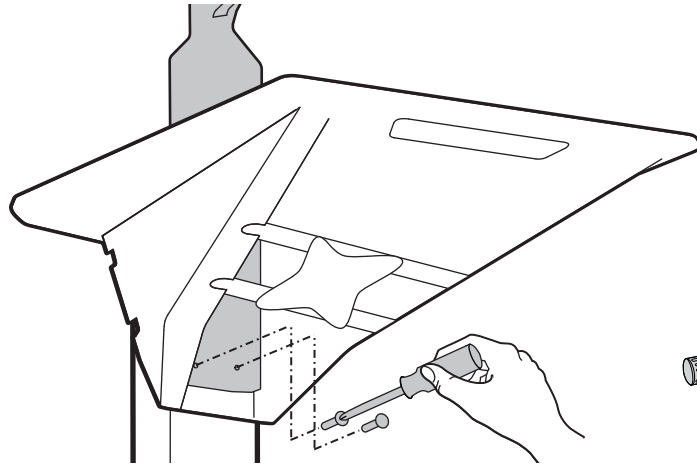
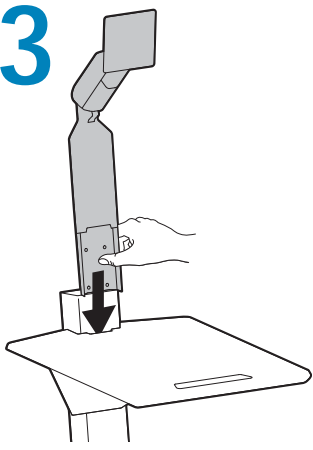
OPTIONAL: Security Screw. 5-sided Security Wrench  
Opcional: Tornillo de seguridad. Llave de seguridad pentagonal  
Facultatif: Vis de sécurité. Clef Allen de sécurité  
Optional: Sicherheitsschraube mit Innenfünfkant. Fünfkant-Inbusschlüssel  
Optioneel: Veiligheidsleutel. 5-kantige veiligheidsleutel  
STE E BA LI: Güvenlik Vidası. 5 Kenarlı Güvenlik Anahtarı  
オプション: 固定ネジ. 5角固定用レンチ  
可选: 安全螺丝. 5角安全扳手

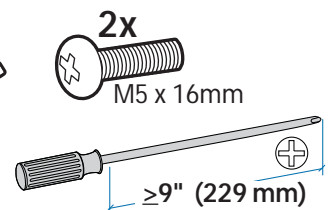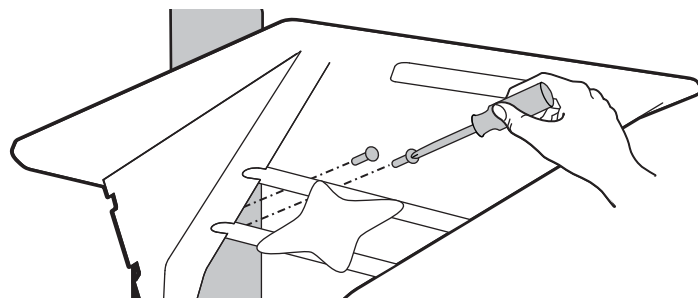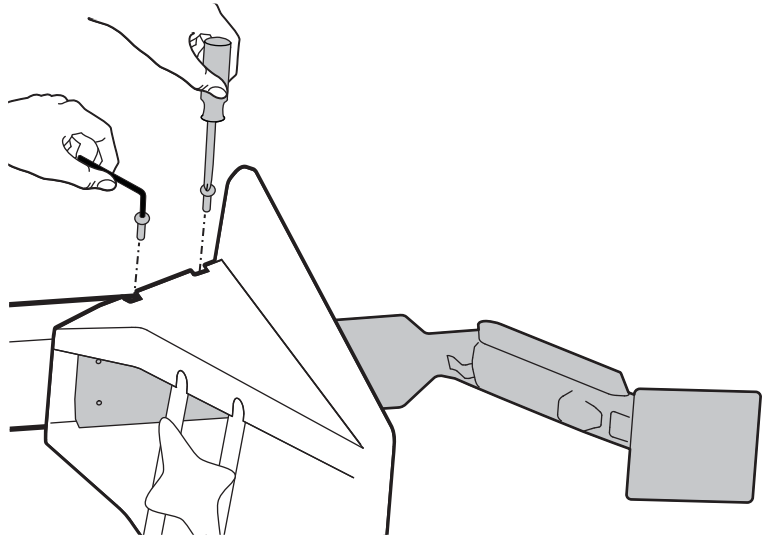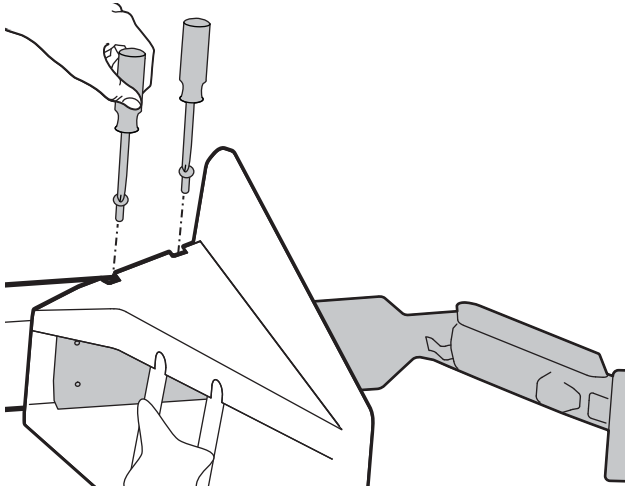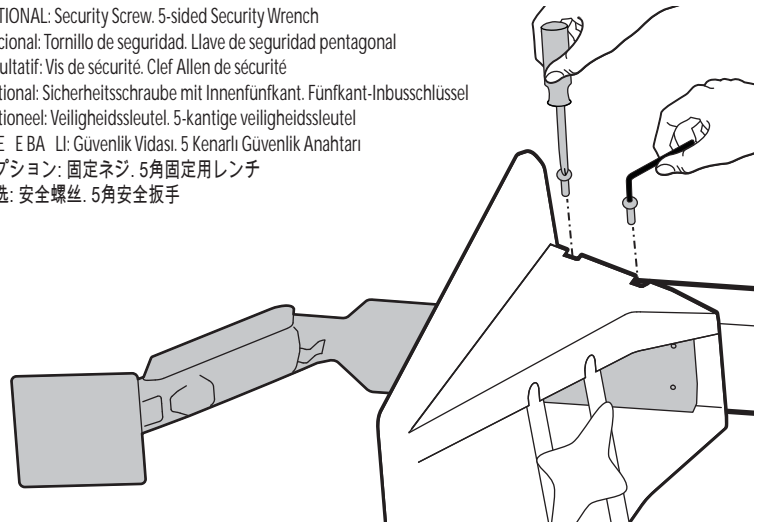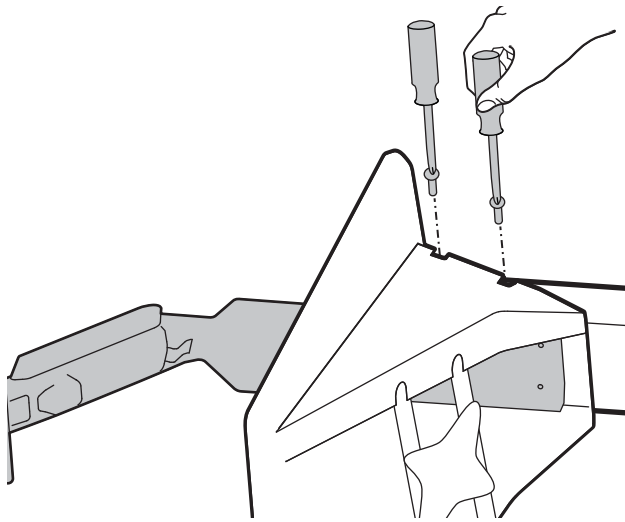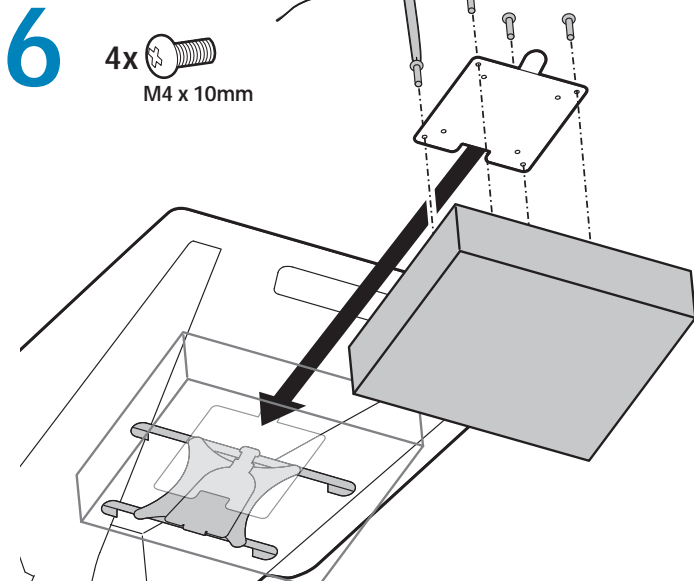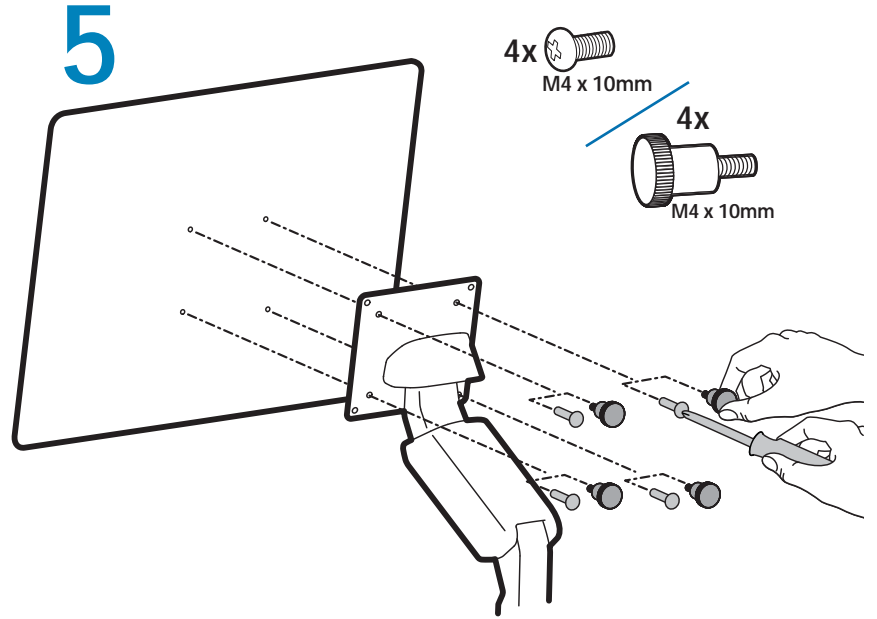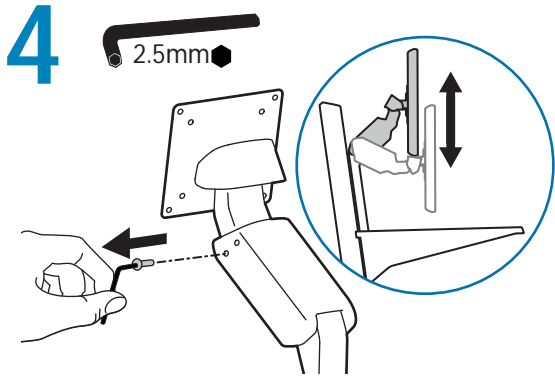
How to remove CPU  
 Cómo quitar la CPU  
 Retrait de l'unité centrale  
 Abnehmen des CPU  
 Het verwijderen van de CPU  
 CPU'nun çıkarılması  
 CPUの取り外し方  
 如何取下 CPU

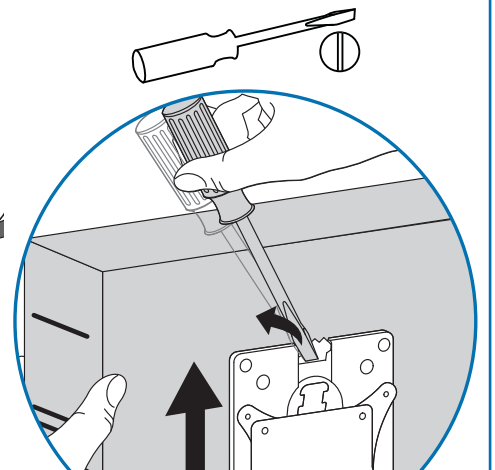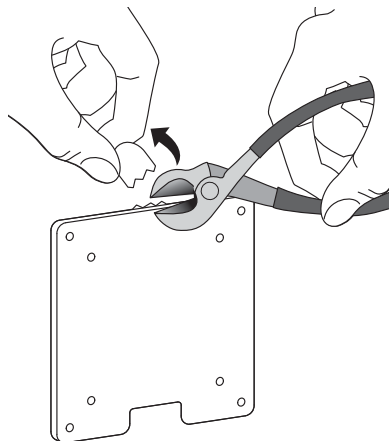
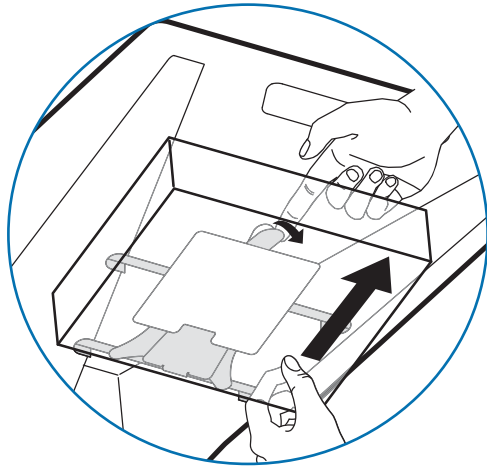
Tamper Proof Option  
 Opción de protección contra manipulación  
 Option d'infraudabilité  
 Manipulationssichere Option  
 Optie diefstalbestendigheid  
 Kurcalamaya Dayanıklı Seçenek  
 タンパープルーフ オプション  
 防盜选件


# 7

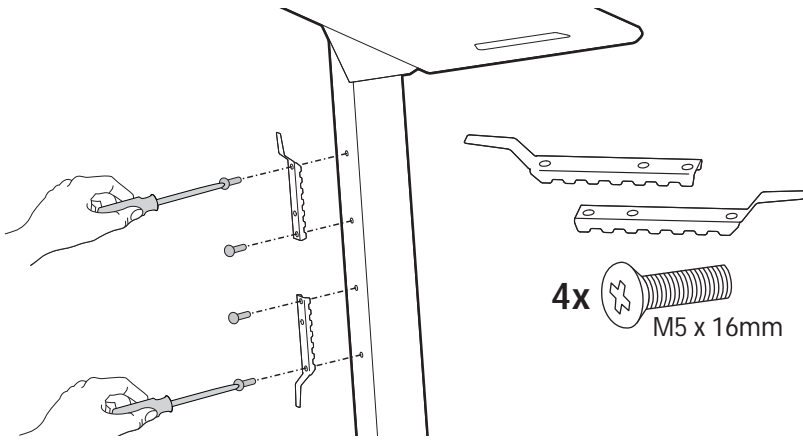
4x  M5 x 16mm

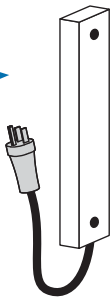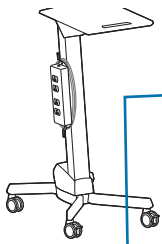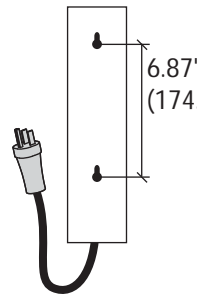
Mounting Holes  
Orificios de montaje  
Trous de montage  
Befestigungslöcher  
Montageopeningen  
Montaj Delikleri  
マウントホール  
安裝孔

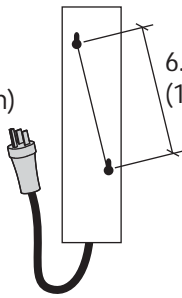6.87"  
(174.6 mm)





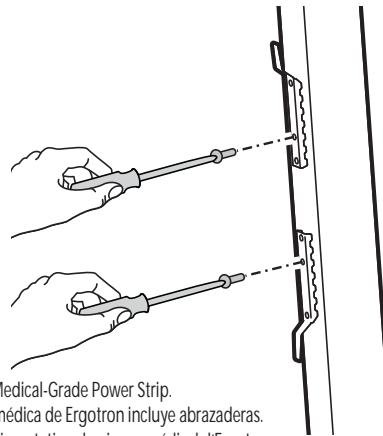
6.87"  
(174.6 mm)


2x  M4 x 8mm

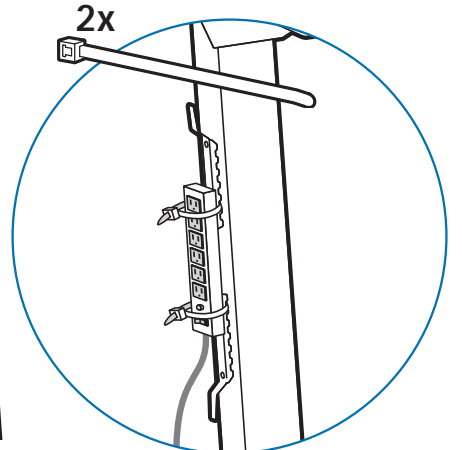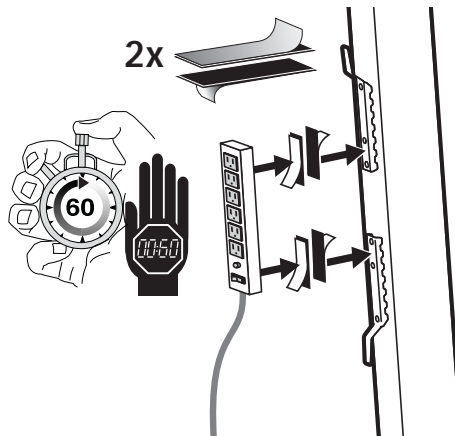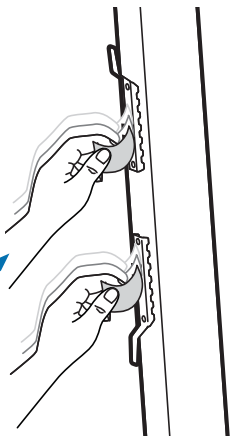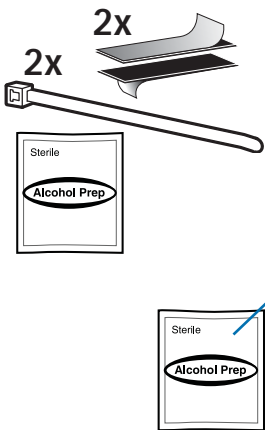
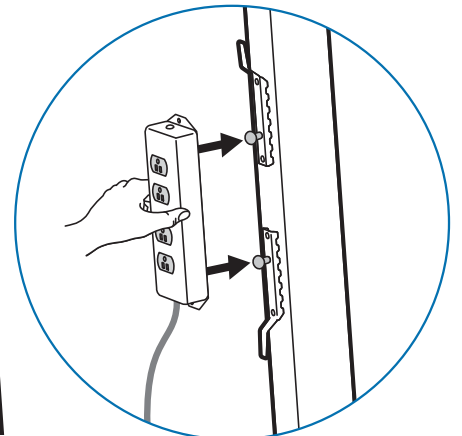
Fasteners supplied with Ergotron Medical-Grade Power Strip.  
La regleta de enchufes de calidad médica de Ergotron incluye abrazaderas.  
Fixations fournies avec le câble d'alimentation de niveau médical d'Ergotron.  
Im Lieferumfang der Ergotron-Steckdosenleiste für den medizinischen Gebrauch enthaltene Befestigungsmittel.  
Bij de Medical Grade Power Strip van Ergotron geleverde bevestigingsmaterialen.  
Ergotron Tibbi Kullanıma Uygun Güç Uzatma Kablosu ile verilen ba lama elemanları.  
エルゴトロン医療レベル パワーストリップに付属のファスナー。  
Ergotron 医用電源插板附带的紧固件。


8

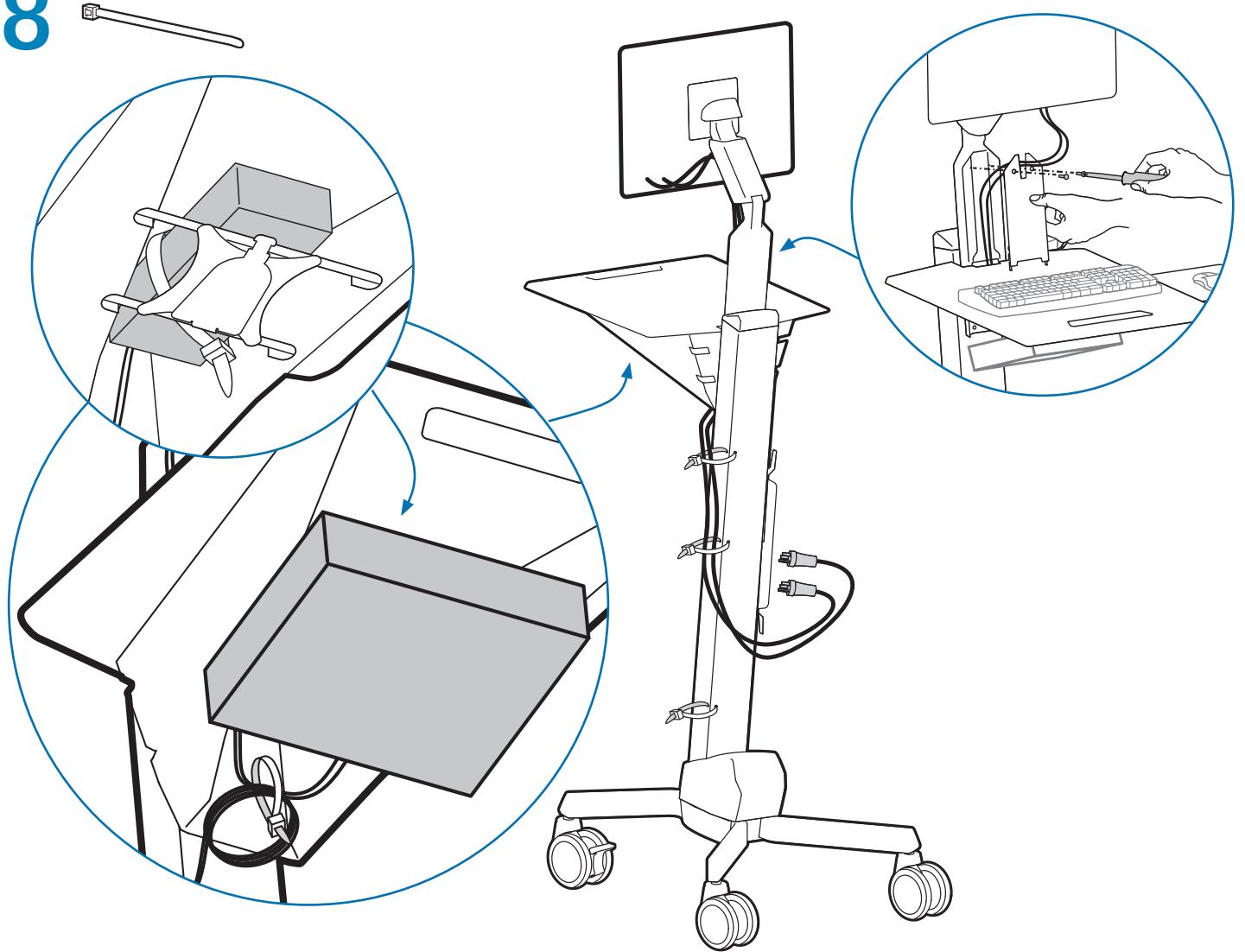9

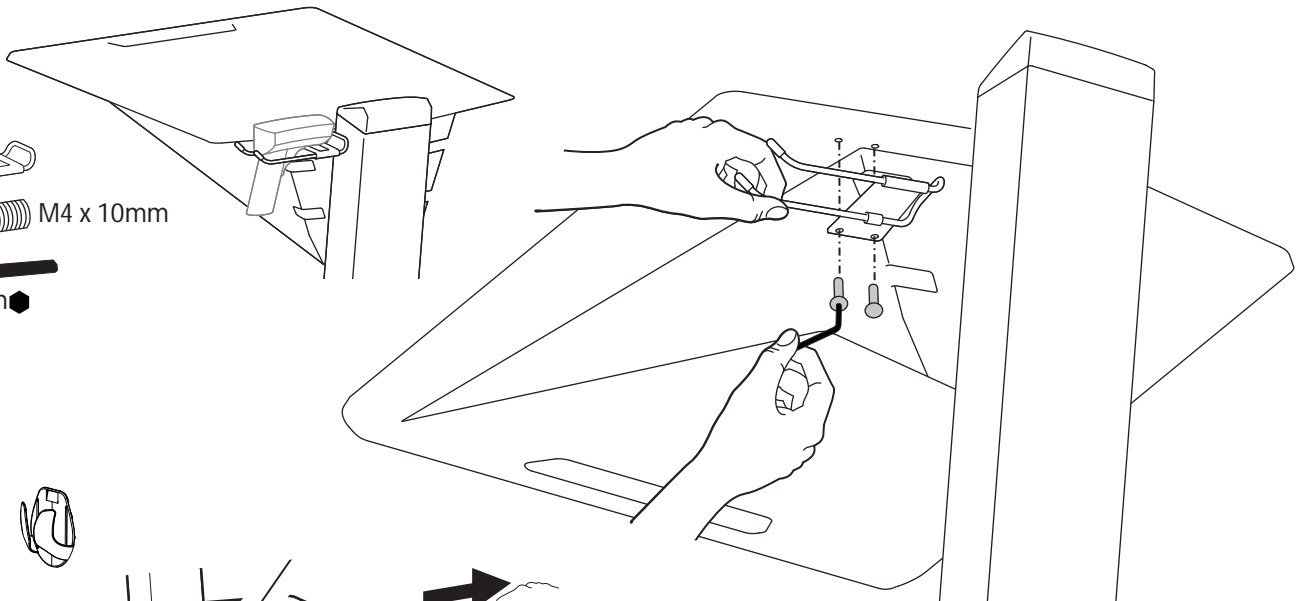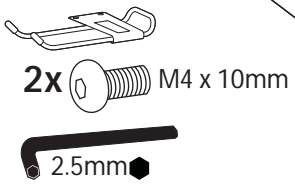
10
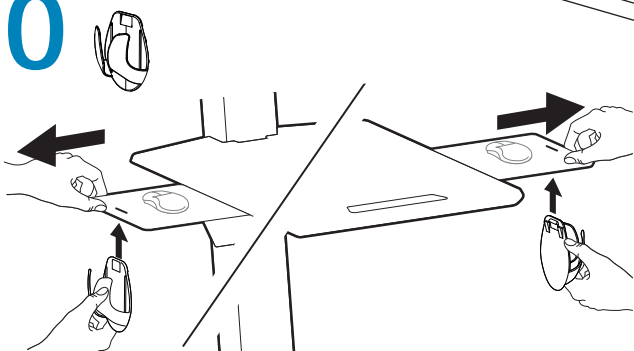
# 11 Adjustment Ajuste Réglage Einstellung Instellen Ayar 調整 调节 조절

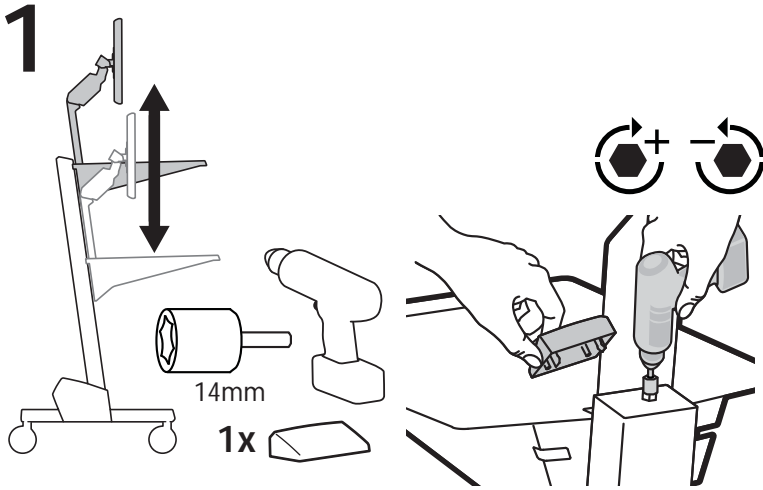
Follow these instructions to tighten or loosen tension.  
 Siga estas instrucciones para ajustar o a lojar la tensión.  
 Suivez ces instructions pour desserrer ou resserrer la tension.  
 Befolgen Sie diese Anleitung, um die Spannung zu lockern oder zu verstärken.  
 Volg deze instructies om de spanning te vergroten of verkleinen.  
 Gerilimi artırmak ya da azaltmak için aşağıdaki talimatları uygulayın.  
 圧力を増減するには次の手順に従います。  
 遵照这些说明拧紧或拧松。  
 다음 지침에 따라 장력을 높이거나 줄이십시오.

**! WARNING**


**Impact Hazard!**  
**Moving Parts can Crush and Cut.**  
 Raise to the full up position before removing equipment.  
 Always adjust the lift tension to accommodate applied load.  
 DO NOT add weight below minimum or above maximum weight capacities.  
 Failure to heed this warning may result in serious personal injury or property damage.

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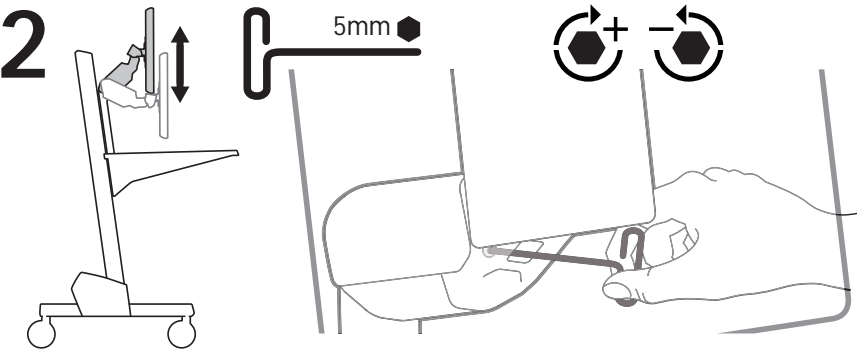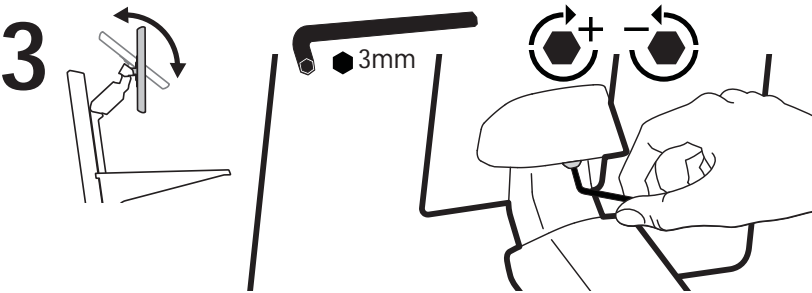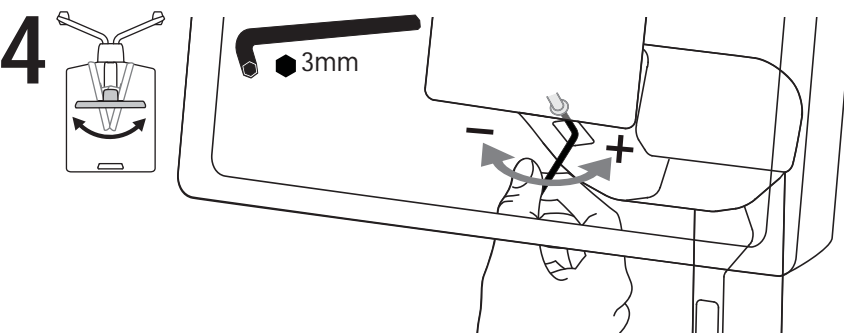
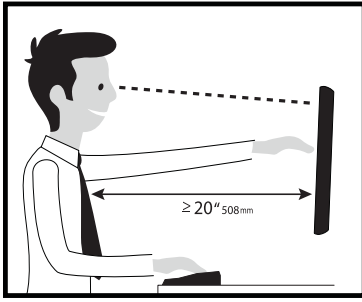
Ready to get moving on an active workstyle? Check out [www.ergotron.com/tips](http://www.ergotron.com/tips) to get started.

Learn more about ergonomic computer use at: [www.ergotron.com/ergonomics](http://www.ergotron.com/ergonomics)

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© 2014 Ergotron, Inc. All rights reserved. NeoFlex™ is a registered trademark of Ergotron, Inc.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA. Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.